





## **MCT® Unit**

### **QUICK INSTALLATION GUIDE**



## PRODUCT DESCRIPTION

- 1 MCT® Unit
- 2 MCT® Kit®
- 3 MCT® Kit® compartment
- 4 Touch screen
- 5 Power switch
- 6 USB conector
- 7 Power cable connection

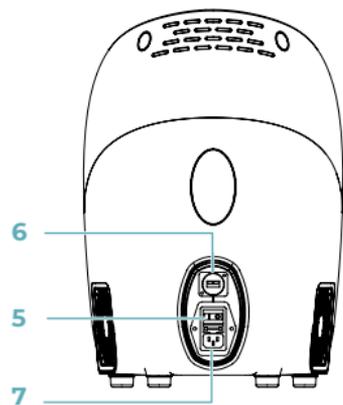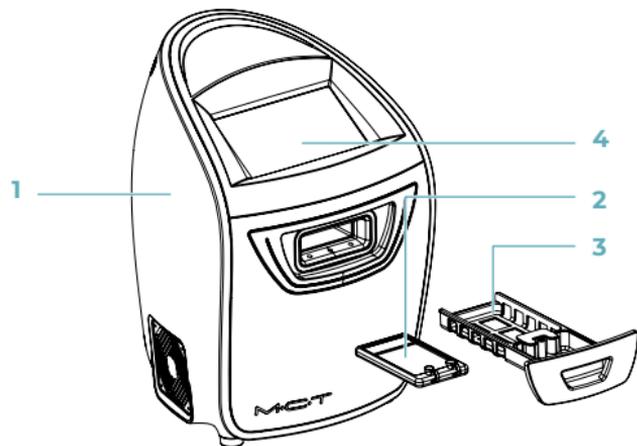


## DEVICE

The MCT® System consists of two devices:


### MCT® UNIT

Activates biological autologous material through the controlled emission of electromagnetic and thermal energy.

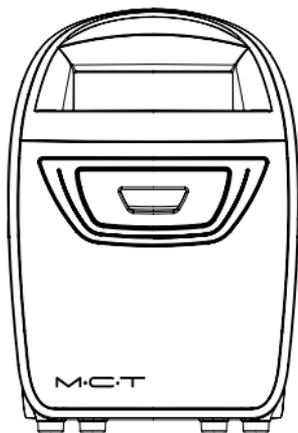


### MCT® KIT®

Device that stores the autologous material to be activated. Patented and developed with geometry and chemical composition specific to the effective photothermal activation of autologous materials.

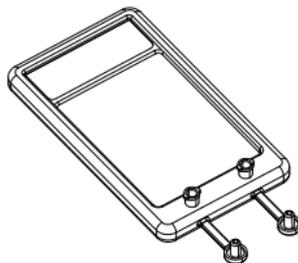


## PROCEDURE


1 Switch on the MCT® Unit

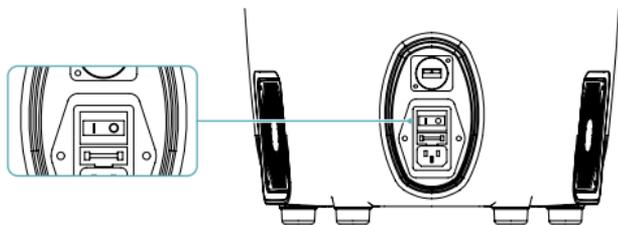


2 Once started, the software is loaded and a start screen appears where the treatment can be selected.

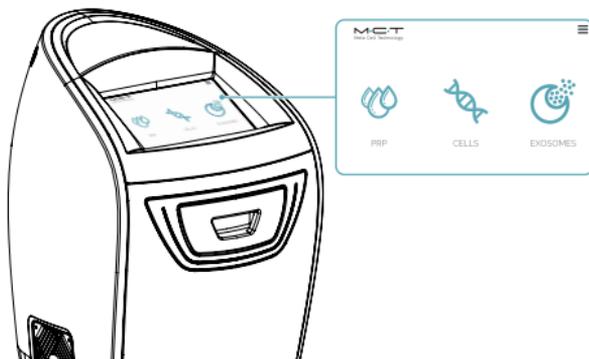


3 Open the MCT Kit's compartment

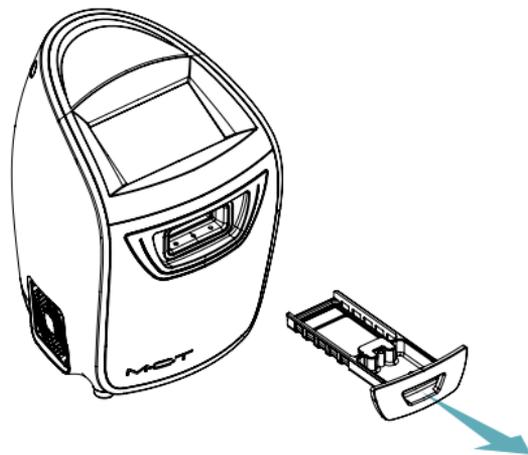

- 4 Load the MCT Kit  
(see instructions for the MCT Kit)

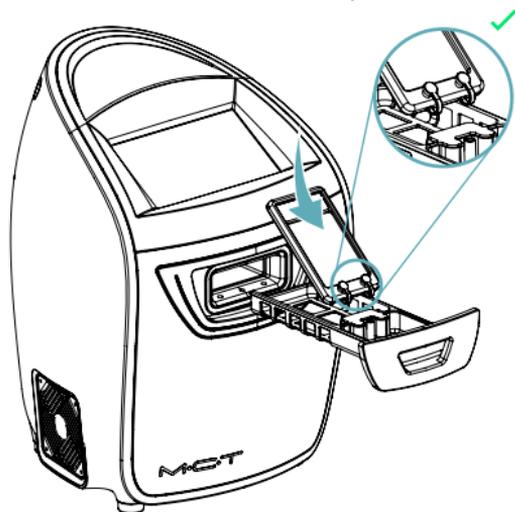

- 5 Close the MCT Kit's compartment

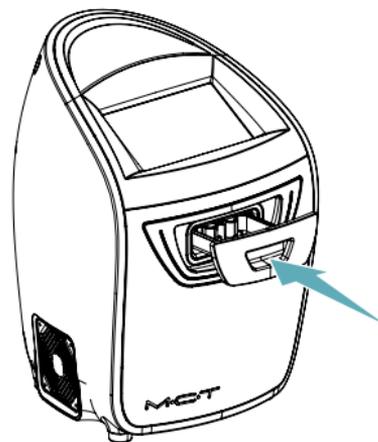

## 6 Select the desired treatment



A



**Preconfigured treatment**

By **selecting a preconfigured treatment**, the amount of electromagnetic and/or thermal energy is already defined, therefore the MCT Unit will change to show the session screen directly.


B

**Custom program**

If you **select the Custom program**, all light and temperature parameters must be set:

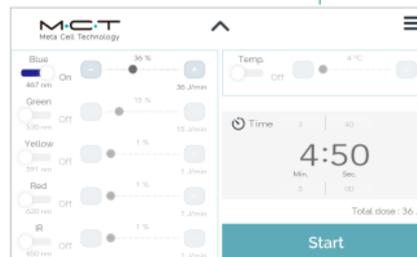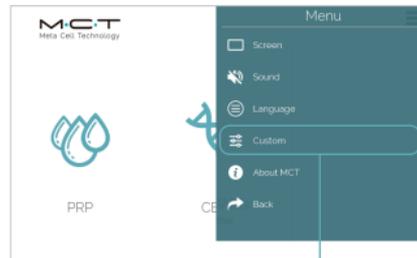

**B**  
**Custom program set up**

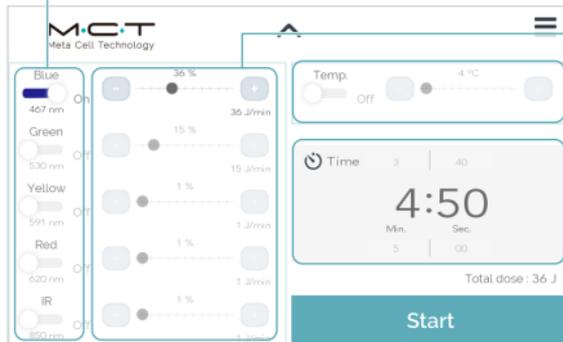

**1 Selector the wavelength**

Select the wavelength or light colour you wish to activate using the activation button



**2 Set the intensity of the wavelength**

Adjust the intensity of the wavelength to be employed using the slider or the +/- buttons



**3 Adjust the temperature**

Adjust the temperature value (slider or +/- buttons)



**4 Set the session time**

Select the session time. Below this is the total energy (in Joules) to be applied in accordance with the settings chosen

## A & B

### 1 Push start button

By **selecting the Start button**, the MCT® UNIT will start the session.

**NOTE** If the MCT Kit's tray is not positioned correctly, a warning screen will appear.

**SOLUTION** Place the tray and select the Start button.



### 3 End of session


At the end of the session, an **acoustic alarm** will inform you that you can remove the MCT Kit from the tray.

**NOTE** The session can be stopped at any time by pressing the STOP button, after which a message asking you to confirm this action will appear.



### 2 Test started

**Once started**, a timer will show the session's remaining time in addition to the settings for all variables.

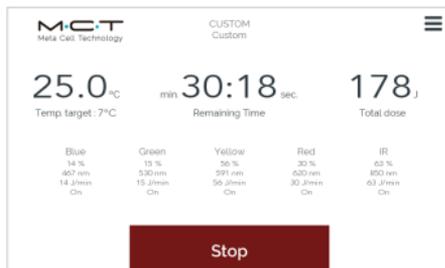

## FURTHER INFORMATION

### Manufacturer

Manufactured by CLINIPRO, S.L.  
Marketed by METACELL TECHNOLOGY, S.L.

### Contact information

Carrer Manel Farrés, 101  
08173 Sant Cugat del Vallès  
Barcelona  
[info@metacelltech.com](mailto:info@metacelltech.com)

### More information

Find online support information by visiting  
[metacelltech.com](http://metacelltech.com).



Full user manual and further information:  
<https://metacelltech.com/manuals/>

[metacelltech.com](http://metacelltech.com)