M.C.T

Meta Cell Technology

User Manual

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0051

Notice

Thank you for your interest in MCT Unit. Meta Cell Technology strives to provide our customers with the highest quality devices, systems, and services.

This manual describes how to operate the MCT Unit system with the MCT Kit consumables. Please thoroughly familiarize yourself with this manual to make the most effective use of the device and the consumables.

Observe all cautionary instructions, safety messages, and warnings that appear in this manual.

Due to continued technological improvements, this manual may not contain the most updated information and is subject to change without prior notice. You can find the latest version of this manual at www.metacelltech.com/manuals.

Copyright

All rights reserved. No parts of the content of this manual may be reproduced or distributed to the public, without the prior written permission of CLINIPRO SL.

EN - User Manual	3
ES - Manual de Usuario	31
FR - Manuel d'Utilisateur	58
DE - Benutzerhandbuch	85
PT - Manual do Usuário	112
IT - Manuale dell'Utente	139
BG - Ръководство за потребителя	166
CS - Uživatelská příručka	193
EL - Εγχειρίδιο Χρήστη	220
HU - Felhasználói kézikönyv	247
LT - Vartotojo vadovas	274
NO - Brukermanual	301
SL - Priročnik za uporabnike	328
SV - Användarmanual	355
TR - Kullanıcı Kılavuzu	382
SK - Používateľská príručka	409

User Manual

Contents

01. SERVICE INFORMATION	6
02. PRODUCT DESCRIPTION	7
03. SYMBOLS	11
04. WARNINGS/ PRECAUTIONS	13
05. TECHNICAL SPECIFICATIONS	15
06. INSTALLATION	16
07. OPERATION	17
08. MAINTENANCE	24
09. CONFIGURATION	26
10. TROUBLESHOOTING	27
11. TERMS OF WARRANTY	29
12. LIFESPAN AND DISPOSAL OF THE PRODUCT	30

01. SERVICE INFORMATION

The company accepts responsibility for the safety, reliability and performance of this equipment only if operational procedures, calibrations and repairs are performed by appropriately qualified persons; if all equipment modifications are authorized in writing by the company and performed by appropriately qualified persons; if the electrical installation of the relevant room complies with all applicable local electrical codes; and if the equipment is used in accordance with the published instructions for use.

If you require technical assistance, please contact your Customer Service Representative or the manufacturer:

CLINIPRO, S.L.

www.metacelltech.com

C/ Manel Farrés, 101

08173 Sant Cugat del Vallès, Spain

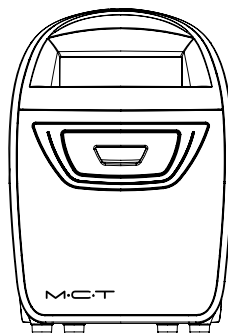
02. PRODUCT DESCRIPTION

Product name: MCT UNIT

Model number: MCT UNIT

MCT UNIT

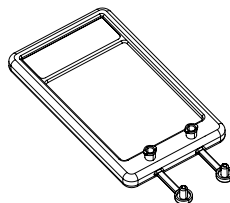
Improves biological autologous material through the controlled emission of electromagnetic energy.

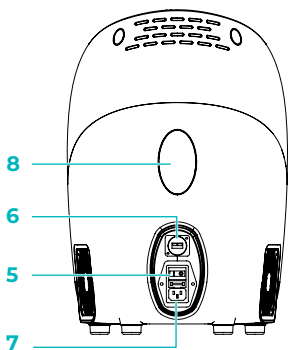
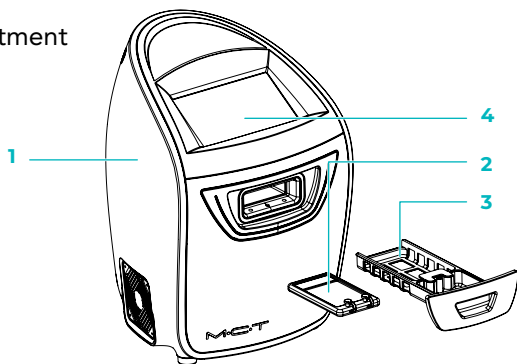
Product used in combination

MCT KIT

Sterile and disposable container for the storage of blood or body fluids. (See User Manual included with the MCT Kit).

- 1 MCT Unit
- 2 MCT Kit
- 3 MCT Kit compartment
- 4 Touch screen

- Power switch 5
- USB connector 6
- Power cable connection 7
- Device label 8

Sample identification device label

Intended purpose

MCT UNIT is intended to be used in combination with the MCT Kit for the rapid obtention of platelet derivatives, such as growth factors and/or biological mediators, through the application of light and temperature.

MCT UNIT is used by physicians for PRP derivatives that will be subsequently re-administered to the patient.

Same as PRP indications such us:

- Osteoarthritis
- Tendinopathies
- Muscle injuries
- Local pain
- Wound healing

Indications of use

The device is indicated for all musculoskeletal or dermatological clinical specialty treatments that require PRP to achieve the stimulation and healing acceleration of tissue.

PRP may be used for:

- Osteoarthritis
- Tendinopathies
- Muscle injuries
- Local pain
- Wound healing

Intended users

MCT UNIT is used by physicians. They can be specialized in traumatology, rheumatology, anesthesiology, rehabilitation, sports medicine, podiatry, dermatology, or any other related medical specialty.

Intended patient population:

Adults, over 18 years, only for medical use.

There is no experience in the use of MCT UNIT during pregnancy and lactation. Therefore, MCT UNIT should not be used if you are pregnant or breastfeeding unless prescribed by your doctor.

(Clinical) Indications:

The device is indicated for all musculoskeletal or dermatological clinical specialty treatments that require PRP to achieve the stimulation and healing acceleration of tissue.

Contraindications:

There are no known contraindications when used as directed. The same as those associated with the related medical procedures.

03. SYMBOLS

The symbols included in the labeling of the product, packaging and IFU are the following

Symbol MANUFACTURER. This symbol is accompanied by the name and the address of the manufacturer

Symbol "DATE OF MANUFACTURE"

Symbol SERIAL NUMBER

Symbol "Model number"

CE symbol with the intervention of the Notified Body IMQ

Symbol for CONSULT INSTRUCTIONS FOR USE

Symbol WEEE

Symbol Medical Device

Symbol "Caution"

Symbol "Risk of electrical shock"

Symbol "UDI"
(01) UDI-DI
(11) PRODUCTION DATE YYMM00
(21) SERIAL NUMBER

Symbol "Alternating current"

Symbol "Fuse"

Symbol "Temperature limit"

Symbol "Keep away from sunlight"

Symbol "Fragile, handle with care"

Symbol "Mandatory use of gloves"

IK06

Symbol "impact rating"

04. WARNINGS / PRECAUTIONS

The warnings are related to the electrical risks for the operator.

The device should only be used by authorized and qualified staff.

The instructions for use of the equipment must be strictly followed for the treatment to be effective.

If there is a power outage or a treatment protocol is interrupted, the sample must be discarded and started over, using a new MCT Kit.

It is mandatory to wear CE marked medical device gloves to operate and clean the equipment.

Never spray water or detergent directly on the MCT Unit.

Do not plug the unit into the wall outlet until the moisture from cleaning has evaporated.

Do not operate any appliance with a power cord other than the official one supplied with the MCT Unit.

MCT UNIT could disturb the operation of electromagnetic fields from other electronic equipment used in diagnostic investigations, evaluations or therapeutic treatments and, in turn, its behavior could be influenced by electromagnetic fields from other equipment.

No medicinal substance, biological material or material containing or consisting of CMR substances is incorporated as an integral part of the medical device.

Do not use any attachments/parts not supplied by the manufacturer. The use of parts not recommended or provided by the manufacturer may cause serious hazardous situations including personal injury.

Do not use containers other than the MCT Kit: the use of other non-original containers prevents photothermal conditioning and therefore makes the treatment useless. In addition, it may damage the unit tray.

05. TECHNICAL SPECIFICATIONS

Technology	Peltier, LED
Compatibility	MCT KIT
Capacity	10ml
Maximum cooling temperature	4°C
Maximum heating temperature	45°C
Maximum air cooling rate	2°C/min
Maximum air heating rate	5°C/min
Temperature uniformity	≤ ±0.5°C
Precision of temperature measurement	≤ ±0.5°C
Temperature control mode	Infrared sensor (contactless)
Increase rate	0.1°C
Refrigeration	Forced air
Tray	Removable and with locking system
Maximum energy emission at 620nm (red)	12 joules/min.
Maximum energy emission at 530nm (green)	10 joules/min.
Maximum energy emission at 467nm (blue)	10 joules/min.
Maximum energy emission at 591nm (amber)	7.5 joules/min.
Maximum energy emission at 850nm (infrared)	10 joules/min.
Radiation emission	Pulsed
Screen	7" colour LCD, capacitive touch screen
CPU	Arm®, Cortex®-M7, 1Gbit NAND. Firmw. Updated by USB 2.0
Approximate dimensions	300mm (l) x 300mm (w) x 400mm (h)
Approximate weight	8kg / 18lb
Power supply	230 VAC, 50 Hz, 150 W*
Fuse	3.15A-250V
Maximum power	150W
Altitude	2000 m
Pollution degree	Type 2
Ambient operating conditions	3.0°C-30.0°C (37°F-86°F) 10% to 85% with no condensation
Storage and transport conditions	0°C-50°C (32°F-122°F)

*Alternative power supply RPS-200-12C 200W EN 60601-1 compliant

06. INSTALLATION

Unpacking

The equipment is supplied in a cardboard box.

Open the shipping carton and remove the protective foam.

Location

MCT Unit should be used correctly placed on a flat surface, ensuring all four rubber feet are in place to allow proper air circulation and enough space to allow easy disconnection of the power cord.

The product must be connected to an electrical installation correctly protected and equipped with a protective earth.

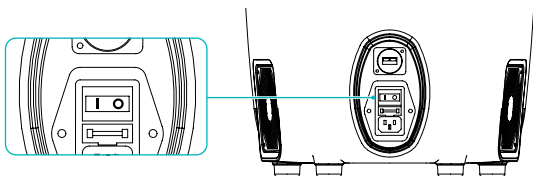
The equipment is designed to be used in basic electromagnetic environments. The standard applied is EN IEC 61326-1:2021 (Electrical equipment for measurement, control and laboratory use - EMC requirements - Part 1: General requirements).

The equipment should only be used by medical personnel who have read and understood this instruction manual or have been adequately qualified in the operation of this equipment. Non-qualified personnel should only use the equipment for occasional maintenance or adjustments, following the indications of the manufacturer's authorized technical service.

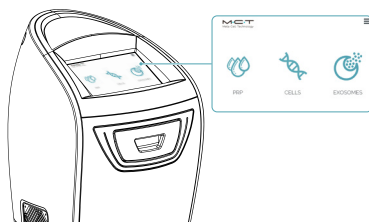
07. OPERATION

Procedure

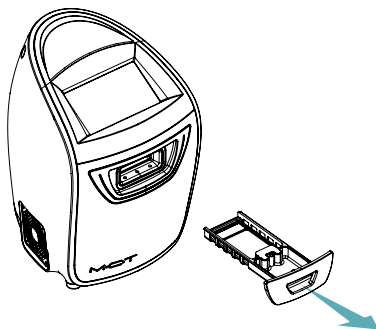
1. Attach the plug and activate the power button located at the rear of the MCT Unit.

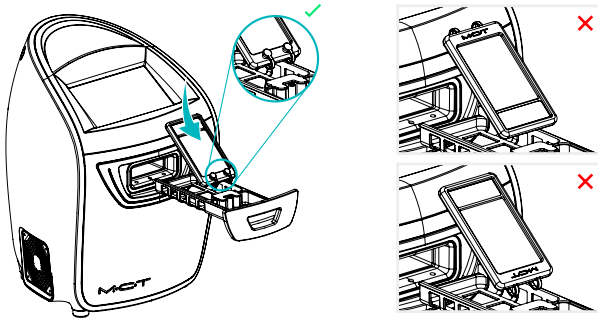
2. The MCT logo and start screen will appear. The user LED light will turn blue, indicating standby mode.

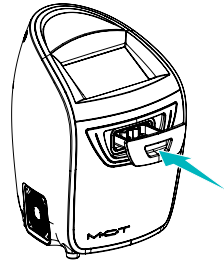
3. Fully open the MCT Kit compartment by inserting your fingers in the hole and pulling.

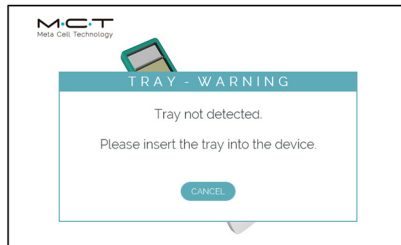
4. Load the MCT Kit as shown in the image.

5. Close the compartment until it clicks shut.

If the MCT Kit tray is not correctly positioned or in its place, a warning screen will indicate that the session cannot be started until the incident is resolved.

6. Select the protocol.

User LED light indicator

LED user
indicator

The user LED is decorative and indicative.

- In standby mode, it shows a blue light.
- During treatment, it displays the protocol's color in a breathing pattern:
 - o RED for PRP & Cells protocols
 - o Blue for Exosomes protocol

Device Protocols

The device has three pre-set automatic protocols. They are individual and to be applied only once with each sample.

Protocol #1. **PRP.**

Photo-thermal stimulation of Platelet Rich Plasma to obtain a higher concentration of ATP and growth factors, with a more prolonged and physiological release.

Stimulus 1 - 620 nm 1 J/cm², 10 min

Stimulus 2 - 4°C, 15 min

Protocol #2. **Cells stimulation.**

Photo-thermal stimulation for metabolism and a greater synthesis of ATP.

Stimulus 1 - 620 nm 1 J/cm², 10 min

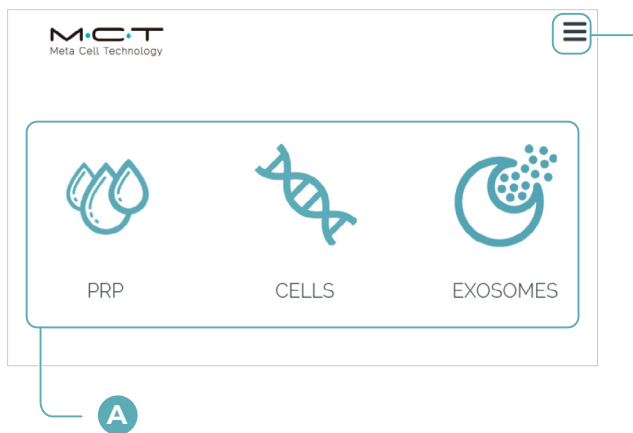
Stimulus 2 - 850 nm 0,5 J/cm², 10 min

Protocol #3. **Exosomes.**

Photo-thermal modulation for the stimulation of exosomes releasing.

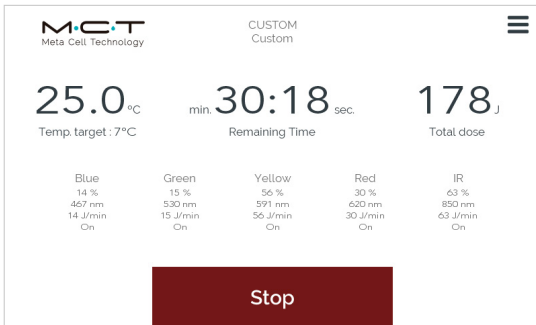
Stimulus 1 – 467 nm 1 J/cm², 10 min

Stimulus 2 - 37°C, 10 min

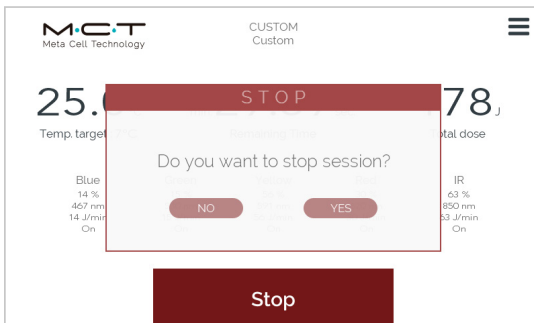
When selecting any of the presets, the session will start automatically and energy parameters will be applied automatically. The MCT UNIT will show the session screen, with the session time and total energy (J).

Once started, a timer will show the session's remaining time and the settings.

A pop-up and an acoustic warning will indicate the session has ended and the MCT KIT may be removed from the MCT UNIT's tray.

In addition, the session may be interrupted at any time by pressing the STOP button, which will display a message asking you to confirm this action.

Once the session ends, open the MCT Kit's compartment and extract the MCT Kit.

If there is a power outage or a treatment protocol is interrupted, the sample must be discarded and started over, using a new MCT Kit.

08. MAINTENANCE

Periodically inspect the TFT screen and fans to prevent dirt/dust to maintain optimal device performance:

Cleaning: Unplug the unit from the wall outlet. Wipe the LCD screen gently with a soft cloth. When cleaning the body, wipe gently with a soft cloth lightly dampened in pure water. Stubborn stains may be removed with a solution of mild soap and warm water.

To clean the fans, suck the dust with an electronic vacuum cleaner or carefully wipe the outer blades with a damp cloth.

Note: Only the external portion of the screen is to be cleaned, as required, by the customer. The unit should never be opened by the customer.

User is responsible of proper decontamination in the event of spillage of the sample or other hazardous material on the equipment. Cleaning and disinfection must be done right away. It may drip to the bottom of the screen or exterior parts and enter the unit and may cause damage to the component parts.

User is expected not to use decontamination or cleaning agents that could cause a hazard as a result of a reaction with parts of the equipment or with material contained in it.

The manufacturer or its agent should be consulted if there is any doubt about the compatibility of decontamination or cleaning agents.

Using a disinfecting towelette or a clean, low-linting cloth dipped in disinfectant solution, gently wipe down the surface of the MCT Unit. Ensure all surfaces remain visibly wet for the contact time recommended by the disinfectant manufacturer. If required by the disinfectant manufacturer, rinse per instructions; otherwise, allow to air dry.

Recommended disinfectants with given compatibility with the MCT Unit include quaternary-based compounds, OPA (ortho-phthalaldehyde), and chlorinated solutions (bleach at dilutions of 1:15 to 1:100) that are EPA-registered to achieve intermediate-level disinfection.

Note: In case of failure, check with manufacturer technical service.

Users must not open the device under any circumstances. Unauthorized tampering or opening of the unit will void the warranty. Manipulation of electronic circuits by unauthorized persons may cause damages difficult to repair.

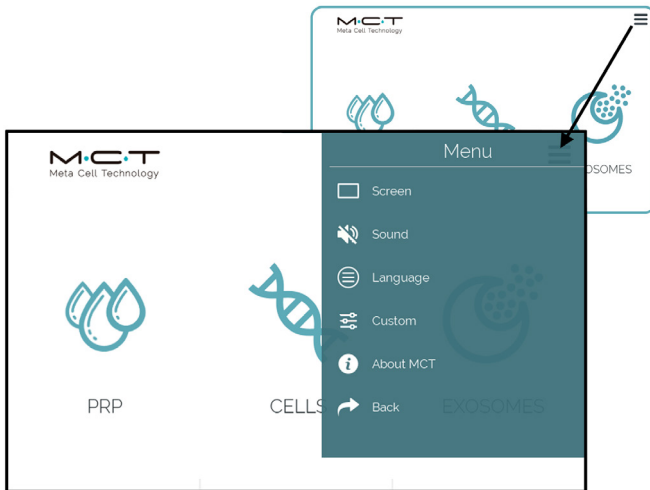
Equipment does not include consumables or spare parts to replace.

Equipment does not require calibration.

MCT UNIT has a USB port for software updates by an authorized professional.

09. CONFIGURATION

The device has a menu where you can configure other interface options. There is an icon in the home screen's top right corner, which will open the following menu when selected:

- 1. Screen:** Allows you to change the screen brightness with a simple intensity slider.
- 2. Sound:** Allows you to change the software's volume.
- 3. Language:** Allows you to choose between English, French and Spanish.
- 4. About MCT:** Information on the software version.

10. TROUBLESHOOTING

The screen may display these errors. Below is a list of errors and their explanations.

If you require technical assistance, please contact your Customer Service Representative or the manufacturer technical service at www.metacelltech.com/contact indicating the error code.

Text screen	DESCRIPTION
0x0005 MCT DRIVERS ERROR	Error initializing drivers for I2C and SMBus Bus
0x0010 BUS I2C HANDLER ERROR	I2C Bus Error
0x0011 I2C BUS RESET ERROR	I2C Bus Reset Error
0x0020 DRV MODULE GENERAL ERROR	Error initializing ICs PCB-DRIVERS-LEDS-PELTIER
0x0021 PWM-DRV MODULE ERROR	Error initializing PWM IC PCB-DRIVERS-LEDS-PELTIER
0x0022 DRV-FANS MODULE ERROR	Error initializing FAN IC PCB-DRIVERS-LEDS-PELTIER
0x0023 DRV-LEDS MODULE ERROR	Error initializing LED IC PCB-DRIVERS-LEDS-PELTIER
0x0024 DRV-PELTIER MODULE ERROR	Error initializing PELTIER IC PCB-DRIVERS-LEDS-PELTIER
0x0025 DRV-PCB TEMPERATURE MODULE ERROR	Error initializing TEMPERATURE IC PCB-DRIVERS-LEDS-PELTIER
0x0026 DRV-LEDS TEMPERATURE MODULE ERROR	Error initializing TEMPERATURE IC LEDs PCB_DRIVERS_LEDS_PELTIER
0x0027 DRV-I MODULE ERROR	Error initializing CURRENT IC PCB-DRIVERS-LEDS-PELTIER
0x0028 DRV-WDT MODULE ERROR	Error initializing WDT IC PCB_DRIVERS_LEDS_PELTIER
0x0029 DRV-RST WDT ERROR	Error resetting WDT IC PCB-DRIVERS-LEDS-PELTIER
0x002a DRV-MUX SMBUS MODULE ERROR	SMBus Error PCB-DRIVERS-LEDS-PELTIER
0x002b DRV-I CALIB-I PELTIER ERROR	PELTIER Calibration Error
0x0030 SMBUS BUS HANDLER ERROR	SMBus Controller Error PCB-DRIVERS-LEDS-PELTIER
0x0031 SMBUS BUS RESET ERROR	SMBus Reset Error PCB-DRIVERS-LEDS-PELTIER
0x0040 TRAY LOCK MODULE ERROR	PCB-BLOCK-TRAY Module Error
0x0041 GPIO TRAY LOCK MODULE ERROR	GPIO Error PCB-BLOCK-TRAY
0x0042 GENERAL ERROR SOLENOID LOCK MODULE	SOLENOID Controller Error PCB-BLOCK-TRAY
0x0043 GENERAL ERROR MODULE TEMPERATURE LOCK	Temperature Sensor Error PCB-BLOCK-TRAY
0x0050 FRONT LEDS MODULE ERROR	Front LEDs Module Error
0x0060 PELTIER RADIATOR TEMPERATURE SENSOR MODULE ERROR	Radiator Temperature Sensor Module Error PELTIER
0x0070 CHAMBER TEMPERATURE SENSOR MODULE ERROR	Fridge/Camera Sensor Module Error
0x0071 CASSETTE TEMPERATURE MODULE ERROR	Cassette Temperature Sensor Module Error
0x0072 CASSETTE HUMIDITY MODULE ERROR	Cassette Humidity Sensor Module Error
0x0073 CASSETTE LIGHT SENSOR MODULE ERROR	Cassette Light Sensor Module Error
0x0080 TFT MODULE ERROR	PCB-TFT Error
0x0090 TFT ILLUMINATION ERROR	TFT Illumination Controller Error PCB-TFT
0x0091 BUZZER ERROR	Buzzer Controller Error PCB-TFT
0x0092 RST WDT-DRV ERROR	I2C WDT Error PCB-DRV
0x0093 FRONT LEDS DRIVER ERROR	I2C Controller Front LEDs Error PCB-LED-FRONTAL
0x0094 FANS DRIVER ERROR	I2C Controller Fan Error PCB-DRIVERS-LEDS-PELTIER
0x0500 TREATMENT TIMER DEVIATION	The process time counter differs by more than 5% from the reference one.

Text screen	DESCRIPTION
0x0095 TRAY LOCK DRIVER ERROR	I2C Controller Tray Lock Error PCB-BLOQUEO-BANDEJA
0x0096 SENSOR TEMP CASSETTE DRIVER ERROR	I2C Controller Cassette Temperature Sensor Error PCB-SENS-NEVERA
0x0097 LED CHAMBER DRIVER ERROR	I2C Fridge LEDs Error PCB-DRIVERS-LEDS-PELTIER
0x0098 PELTIER DRIVER ERROR	PELTIER Controller Error PCB-DRIVERS-LEDS-PELTIER
0x0099 TEMPERATURE DRIVER ERROR	Equipment Temperature Controller Error
0x009a CURRENT SENSOR DRIVER ERROR	Current Sensor Controller Error
0x009b MOISTURE DRIVER ERROR	Humidity Controller Error
0x009c PCB TEMPERATURE DRIVER ERROR	PCBs Temperature Controller Error
0x009d CHAMBER TEMP ERROR	Fridge/Camera Temperature Controller Error
0x009e TRAY SENSORS ERROR	Tray Sensor Error
0x009f EEPROM ERROR	EEPROM Error
0x00a0 LED FAILURE HANDLER ERROR	LEDs Failure Controller Error
0x0100 HIGH TEMPERATURE CASSETTE	High Temperature CASSETTE
0x0101 CHAMBER HIGH TEMPERATURE	High Temperature FRIDGE/CAMERA
0x0102 PELTIER HIGH TEMPERATURE	High Temperature PELTIER
0x0103 HIGH INTERNAL TEMPERATURE	High Temperature EQUIPMENT
0x0180 CASSETTE TEMPERATURE SENSOR FAILURE	Cassette Temperature Sensor Failure
0x0181 CHAMBER TEMPERATURE SENSOR FAILURE	Fridge/Camera Temperature Sensor Failure
0x0182 PELTIER TEMPERATURE FAULT	PELTIER Temperature Sensor Failure
0x0183 INTERNAL TEMPERATURE FAILURE	Equipment Temperature Sensor Failure
0x0200 EXCESS CONSUMPTION POWER CHAMBER FAN	Fridge/Camera Fan Overconsumption
0x0201 PELTIER FAN EXCESS POWER CONSUMPTION	PELTIER Overconsumption
0x0202 CHASSIS FAN EXCESS POWER CONSUMPTION	Chassis Fans Overconsumption
0x0203 PELTIER POWER EXCESS CONSUMPTION	PELTIER Overconsumption
0x0280 CHAMBER FAN FAILURE	Internal Camera/Fridge Fan Failure
0x0281 PELTIER FAN FAILURE	PELTIER Radiator Fan Failure
0x0282 CHASSIS FAN FAILURE	Chassis Fans Failure
0x0283 PELTIER FAULT	PELTIER Not Connected Failure
0x0300 HIGH TEMPERATURE LEDS CHAMBER	High Temperature Fridge/Camera LEDs
0x0301 DRIVER HIGH TEMPERATURE	High Temperature PCB-DRIVERS-LEDS-PELTIER
0x0302 TFT HIGH TEMPERATURE	High Temperature PCB-TFT
0x0303 CPU HIGH TEMPERATURE	High Temperature CPU PCB-TFT
0x0304 TRAY LOCK HIGH TEMPERATURE	High Temperature PCB-BLOCK-TRAY
0x0305 CHAMBER LED DRIVER HIGH TEMPERATURE	High Temperature IC Driver Fridge/Camera LEDs
0x0306 FRONT LED DRIVER HIGH TEMPERATURE	High Temperature IC Driver Front LEDs
0x0380 CHAMBER LEDS TEMPERATURE FAULT	Fridge/Camera LEDs Temperature Sensor Failure
0x0381 DRIVER TEMPERATURE FAULT	PCB-DRIVERS-LEDS-PELTIER Temperature Sensor Failure
0x0382 TFT TEMPERATURE FAULT	PCB-TFT Temperature Sensor Failure
0x0383 CPU TEMPERATURE FAULT	CPU PCB-TFT Temperature Sensor Failure
0x0384 TRAY LOCK TEMPERATURE FAULT	Tray Lock Temperature Sensor Failure PCB-BLOQUEO-BANDEJA
0x0400 CHAMBER RED LED FAILURE	Fridge/Camera RED LED Failure
0x0401 CHAMBER GREEN LED FAILURE	Fridge/Camera GREEN LED Failure
0x0402 CHAMBER BLUE LED FAILURE	Fridge/Camera BLUE LED Failure
0x0403 CHAMBER YELLOW LED FAILURE	Fridge/Camera YELLOW LED Failure
0x0404 CHAMBER INFRARED LED FAILURE	Fridge/Camera INFRARED LED Failure
0x0480 FRONT RED LED FAILURE	Front RED LED Failure
0x0481 FRONT GREEN LED FAILURE	Front GREEN LED Failure
0x0482 FRONT BLUE LED FAILURE	Front BLUE LED Failure
0x0500 TREATMENT TIMER DEVIATION	The process time counter differs by more than 5% from the reference one.

11. TERMS OF WARRANTY

MCT Unit is covered by a warranty lasting 2 years from the date of purchase on the fiscal document delivery (accompanying document confirming the date of purchase) and / or invoice.

Warranty is limited to replacement or repair of individual components or pieces with manufacturing defects.

The guarantee does not cover:

- Malfunction or damage due to negligence in handling or error on the part of the customer
- Malfunction or damage due to third party repair or modification
- Malfunction or damage due to the use of the product with a device not authorized by Clinipro
- Malfunction or damage due to use of the product for a purpose other than its intended purpose
- Malfunction or damage caused by natural disasters.

Be sure to read the entire operation manual before using the product.

The warranty is void if the medical device has been opened, tampered with, or repaired by unauthorized personnel. Defective devices must be reported and returned to an authorized distributor.

Any serious incident related to the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient are established.

12. LIFESPAN AND DISPOSAL OF THE PRODUCT

MCT Unit has a lifespan of at least 9 years, provided it is used under the stated operating conditions and specified maintenance is performed.

MCT Unit is an electrical/electronic device and must therefore be disposed of properly. Consult a local waste management center.

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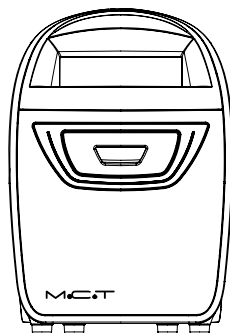
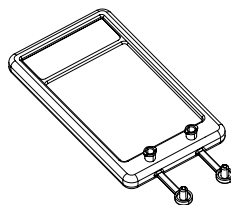
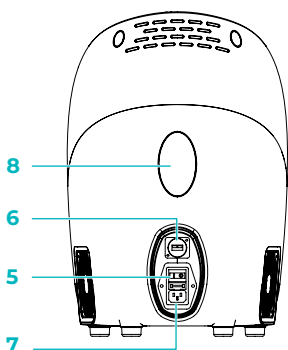
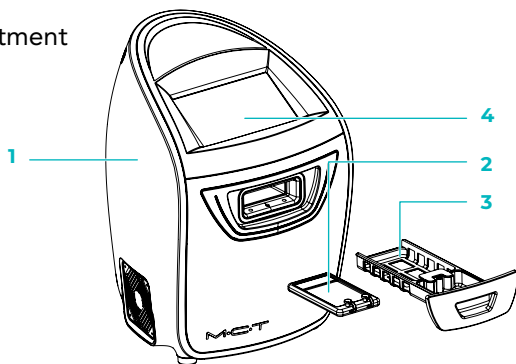
- 1 MCT Unit
- 2 MCT Kit
- 3 MCT Kit compartment
- 4 Touch screen

- Power switch 5
- USB conector 6
- Power cable connection 7
- Device label 8

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Text screen	DESCRIPTION
0x0005 MCT DRIVERS ERROR	Error initializing drivers for I2C and SMBus Bus
0x0010 BUS I2C HANDLER ERROR	I2C Bus Error
0x0011 I2C BUS RESET ERROR	I2C Bus Reset Error
0x0020 DRV MODULE GENERAL ERROR	Error initializing ICs PCB-DRIVERS-LEDS-PELTIER
0x0021 PWM-DRV MODULE ERROR	Error initializing PWM IC PCB-DRIVERS-LEDS-PELTIER
0x0022 DRV-FANS MODULE ERROR	Error initializing FAN IC PCB-DRIVERS-LEDS-PELTIER
0x0023 DRV-LEDS MODULE ERROR	Error initializing LED IC PCB-DRIVERS-LEDS-PELTIER
0x0024 DRV-PELTIER MODULE ERROR	Error initializing PELTIER IC PCB-DRIVERS-LEDS-PELTIER
0x0025 DRV-PCB TEMPERATURE MODULE ERROR	Error initializing TEMPERATURE IC PCB-DRIVERS-LEDS-PELTIER
0x0026 DRV-LEDS TEMPERATURE MODULE ERROR	Error initializing TEMPERATURE IC LEDS PCB_DRIVERS_LEDS_PELTIER
0x0027 DRV-I MODULE ERROR	Error initializing CURRENT IC PCB-DRIVERS-LEDS-PELTIER
0x0028 DRV-WDT MODULE ERROR	Error initializing WDT IC PCB_DRIVERS_LEDS_PELTIER
0x0029 DRV-RST WDT ERROR	Error resetting WDT IC PCB-DRIVERS-LEDS-PELTIER
0x002a DRV-MUX SMBUS MODULE ERROR	SMBus Error PCB-DRIVERS-LEDS-PELTIER
0x002b DRV-I CALIB-I PELTIER ERROR	PELTIER Calibration Error
0x0030 SMBUS BUS HANDLER ERROR	SMBus Controller Error PCB-DRIVERS-LEDS-PELTIER
0x0031 SMBUS BUS RESET ERROR	SMBus Reset Error PCB-DRIVERS-LEDS-PELTIER
0x0040 TRAY LOCK MODULE ERROR	PCB-BLOCK-TRAY Module Error
0x0041 GPIO TRAY LOCK MODULE ERROR	GPIO Error PCB-BLOCK-TRAY
0x0042 GENERAL ERROR SOLENOID LOCK MODULE	SOLENOID Controller Error PCB-BLOCK-TRAY
0x0043 GENERAL ERROR MODULE TEMPERATURE LOCK	Temperature Sensor Error PCB-BLOCK-TRAY
0x0050 FRONT LEADS MODULE ERROR	Front LEDs Module Error
0x0060 PELTIER RADIATOR TEMPERATURE SENSOR MODULE ERROR	Radiator Temperature Sensor Module Error PELTIER
0x0070 CHAMBER TEMPERATURE SENSOR MODULE ERROR	Fridge/Camera Sensor Module Error
0x0071 CASSETTE TEMPERATURE MODULE ERROR	Cassette Temperature Sensor Module Error
0x0072 CASSETTE HUMIDITY MODULE ERROR	Cassette Humidity Sensor Module Error
0x0073 CASSETTE LIGHT SENSOR MODULE ERROR	Cassette Light Sensor Module Error
0x0080 TFT MODULE ERROR	PCB-TFT Error
0x0090 TFT ILLUMINATION ERROR	TFT Illumination Controller Error PCB-TFT
0x0091 BUZZER ERROR	Buzzer Controller Error PCB-TFT
0x0092 RST WDT-DRV ERROR	I2C WDT Error PCB-DRV
0x0093 FRONT LEADS DRIVER ERROR	I2C Controller Front LEDs Error PCB-LED-FRONTAL
0x0094 FANS DRIVER ERROR	I2C Controller Fan Error PCB-DRIVERS-LEDS-PELTIER
0x0500 TREATMENT TIMER DEVIATION	The process time counter differs by more than 5% from the reference one.

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Text screen	DESCRIPTION
0x0005 MCT DRIVERS ERROR	Error initializing drivers for I2C and SMBus Bus
0x0010 BUS I2C HANDLER ERROR	I2C Bus Error
0x0011 I2C BUS RESET ERROR	I2C Bus Reset Error
0x0020 DRV MODULE GENERAL ERROR	Error initializing ICs PCB-DRIVERS-LEDS-PELTIER
0x0021 PWM-DRV MODULE ERROR	Error initializing PWM IC PCB-DRIVERS-LEDS-PELTIER
0x0022 DRV-FANS MODULE ERROR	Error initializing FAN IC PCB-DRIVERS-LEDS-PELTIER
0x0023 DRV-LEDS MODULE ERROR	Error initializing LED IC PCB-DRIVERS-LEDS-PELTIER
0x0024 DRV-PELTIER MODULE ERROR	Error initializing PELTIER IC PCB-DRIVERS-LEDS-PELTIER
0x0025 DRV-PCB TEMPERATURE MODULE ERROR	Error initializing TEMPERATURE IC PCB-DRIVERS-LEDS-PELTIER
0x0026 DRV-LEDS TEMPERATURE MODULE ERROR	Error initializing TEMPERATURE IC LEDs PCB_DRIVERS_LEDS_PELTIER
0x0027 DRV-I MODULE ERROR	Error initializing CURRENT IC PCB-DRIVERS-LEDS-PELTIER
0x0028 DRV-WDT MODULE ERROR	Error initializing WDT IC PCB_DRIVERS_LEDS_PELTIER
0x0029 DRV-RST WDT ERROR	Error resetting WDT IC PCB-DRIVERS-LEDS-PELTIER
0x002a DRV-MUX SMBUS MODULE ERROR	SMBus Error PCB-DRIVERS-LEDS-PELTIER
0x002b DRV-I CALIB-I PELTIER ERROR	PELTIER Calibration Error
0x0030 SMBUS BUS HANDLER ERROR	SMBus Controller Error PCB-DRIVERS-LEDS-PELTIER
0x0031 SMBUS BUS RESET ERROR	SMBus Reset Error PCB-DRIVERS-LEDS-PELTIER
0x0040 TRAY LOCK MODULE ERROR	PCB-BLOCK-TRAY Module Error
0x0041 GPIO TRAY LOCK MODULE ERROR	GPIO Error PCB-BLOCK-TRAY
0x0042 GENERAL ERROR SOLENOID LOCK MODULE	SOLENOID Controller Error PCB-BLOCK-TRAY
0x0043 GENERAL ERROR MODULE TEMPERATURE LOCK	Temperature Sensor Error PCB-BLOCK-TRAY
0x0050 FRONT LEADS MODULE ERROR	Front LEDs Module Error
0x0060 PELTIER RADIATOR TEMPERATURE SENSOR MODULE ERROR	Radiator Temperature Sensor Module Error PELTIER
0x0070 CHAMBER TEMPERATURE SENSOR MODULE ERROR	Fridge/Camera Sensor Module Error
0x0071 CASSETTE TEMPERATURE MODULE ERROR	Cassette Temperature Sensor Module Error
0x0072 CASSETTE HUMIDITY MODULE ERROR	Cassette Humidity Sensor Module Error
0x0073 CASSETTE LIGHT SENSOR MODULE ERROR	Cassette Light Sensor Module Error
0x0080 TFT MODULE ERROR	PCB-TFT Error
0x0090 TFT ILLUMINATION ERROR	TFT Illumination Controller Error PCB-TFT
0x0091 BUZZER ERROR	Buzzer Controller Error PCB-TFT
0x0092 RST WDT-DRV ERROR	I2C WDT Error PCB-DRV
0x0093 FRONT LEADS DRIVER ERROR	I2C Controller Front LEDs Error PCB-LED-FRONTAL
0x0094 FANS DRIVER ERROR	I2C Controller Fan Error PCB-DRIVERS-LEDS-PELTIER
0x0500 TREATMENT TIMER DEVIATION	The process time counter differs by more than 5% from the reference one.

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